

3M™ Ioban™ 2 Antimicrobial Incise Drape



Indication for Use

- 3M Ioban 2 Antimicrobial Incise Drape is indicated for use as an incise drape with continuous antimicrobial activity. It is intended for external use only.

Contraindication

- Do not use 3M Ioban 2 Antimicrobial Incise Drape on patients with known sensitivity to iodine.

Warning

- Do not defibrillate through plastic drape as arcing, ineffective defibrillation, or patient injury may occur.
- Do not re-sterilize.
- **Maximum vacuum pressure limit for package is 15 inHgA.** Only 3M Ioban 2 Antimicrobial Incise Drapes individually packaged in foil can be ethylene oxide (EO) sterilized. EO gas can adversely interact with iodine. Contact 3M for further information on the above limitation.

Directions for Use

1. Disinfect the operative site according to the health care facility policy and allow the prepped skin to dry.
2. Hold the drape with the printing on the handle facing up, and separate the printed handle from the white handle. Position hands at the outer edges of the handles for best results.
 - Printed handle is the 3M Ioban 2 Antimicrobial Incise Drape side.
 - White handle is the liner side.
3. While one person holds the printed handle, the other person holds the white handle to begin removing the liner (see figure a).
4. Position the drape on patient to cover the prepped surgical site, begin removing the liner.
5. Using a sterile towel, press down firmly on the film, first contacting the skin along the intended incision line to ensure good skin contact. Firmly smooth film into place **WITHOUT TENSION**, working away from the intended incision line to achieve wrinkle free adhesion (see figure b).
6. Remove the rest of the liner and discard.
7. If desired, remove the printed handle:
 - Place thumb on the 3M Ioban 2 Antimicrobial Incise Drape film next to handle and begin tearing the handle with the other hand (see figure c).
- B. To remove drape, gently peel back drape at 180-degree angle from the skin. This means folding the film back on itself and maintaining that angle while pulling the film away from the skin (see figure d and e). During the removal process hold onto the film with the thumb and hand as close as possible to the junction of skin and adhesive film. This prevents severe stretching. Skin of elderly patients is typically fragile and thus requires even greater care when applying and removing adhesive products such as drapes.

Storage Conditions


For best results, store at room temperature. Avoid excessive heat and humidity.

For further information, please contact your local 3M representative or contact us at 3M.com and select your country.


Explanation of Symbols


 Catalogue Number

 Not Made With Natural Rubber Latex

 Caution, see instructions for use

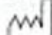
 Do not use if package is damaged

 Do not reuse

 Use by date

 Batch code

 Manufacturer

 Date of manufacture

 Sterilized using irradiation

CE 0086

Made in Poland

3M Health Care
2510 Conway Ave.
St. Paul, MN 55144
800-228-3957
3M.com/Medical

3M Deutschland GmbH
Health Care Business
Carl-Schurz-Str. 1
41453 Neuss
Germany

3M and Ioban are trademarks of 3M.

Used under license in Canada.

© 2017, 3M. All rights reserved.

Issue Date: 2017-03
YP-2763-0360-6/0317
3010018926/0317

