



Slip the cording over the forceps.

**⚠ Do not stretch the cording more than necessary. This may cause the cording to break.**


With another forceps clamp the umbilical cord close to the mother.

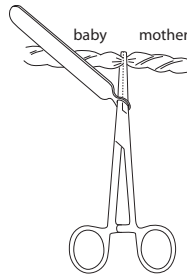


Use the cording forceps to clamp the umbilical cord close to the baby. Ensure that the end tips of the forceps do not project past the diameter of the umbilical cord.

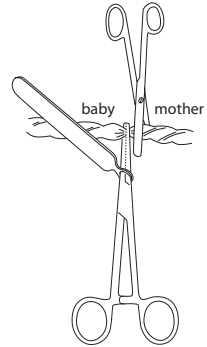


Cut through the umbilical cord on the mother's side close to the forceps with the cording.

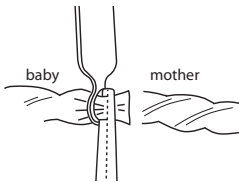

Securely grasp the handle of the cording and pull the cording from the forceps and over the umbilical cord.

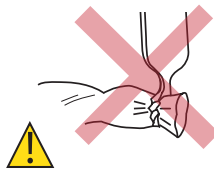

**⚠ Do not twist the cording as shown here. This may cause the cord ring to break.**

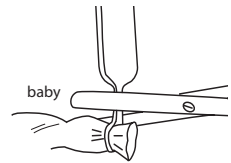

Cut off the handle of the cording minimum 1 mm from the ring of the cording which now is sitting around the umbilical cord.

Read the user manual	Manufacturer	Do not reuse. Disposable Product.
<b>REF</b> Article number	Use before	Medical Device Directive 93/42/EEC
<b>LOT</b> Lot number	Do not use broken packaging.	<b>STERILE EO</b> Sterilized with ethylene oxide.

REF: 9607 och 9607S

40-9607/9607S-001-201021

Kvalitet ISO 9001 Miljö ISO 14001 Medical ISO 13485

**Wing Plast** - Quality Medical Instruments

Wing Plast AB, Nitgatan 11, 333 33 Smålandsstenar, 0371 330 30, info@wingplast.se, www.wingplast.se

**Wing Plast AB**