

DASH 6™ BLUNT DRAWING UP FILTER NEEDLE

Instructions For Use

Please read these instructions before use.



INTENDED USE

The DASH 6™ Blunt Drawing Up Filter Needle is intended to be used with ISO 80369-6 NRFit® neuraxial syringes for the drawing up of neuraxial medication and anaesthetic.

INDICATION

The device is used with an ISO 80369-6 NRFit® compliant neuraxial syringe and is used for drawing up and filtering neuraxial medication and anaesthetic for subsequent administration to a patient.

INTENDED USER

The device is intended to be used by a trained healthcare professional.

CONTRAINDICATIONS

The device is not intended for parenteral or enteral use.

WARNINGS & PRECAUTIONS

Do not use if packaging is damaged.

Do not reuse or reprocess the device.

Do not use if the tip is visibly blocked or if the device is damaged.

The device is single use only. Reuse may result in loss of device performance and/or infection.

DIRECTIONS

Always follow the hospital protocol when using the device.

The device is to be used in conjunction with the ISO 80369-6 NRFit® syringe. Please follow the instructions for use as per the ISO 80369-6 NRFit® syringe provided for drawing up of neuraxial medication and anaesthetic.

Attach the device to an ISO 80369-6 NRFit® syringe by twisting the hub of the blunt drawing up filter needle onto the ISO 80369-6 NRFit® compliant syringe connector.

STORAGE CONDITION

Keep dry and away from direct sunlight.

DISPOSAL

Dispose of the device as per hospital protocol.



AVAILABLE PRODUCTS

DESCRIPTION

15291050 DASH 6™ ISO 80369-6 NRFit®
Compliant Blunt Drawing Up
Filter Needle
5µm, 18G x 50mm



GBUK Group Ltd.
Woodland House,
Blackwood Hall Business Park,
North Duffield, Selby,
North Yorkshire, YO8 5DD, UK.

GBUK Healthcare is a trading name of **GBUK Group Ltd.**

www.gbukgroup.com

CE 1639



STERILE EO

Rx Only

Caution: Federal Law (USA)
restricts this device to sale by
or on the order of a physician.

15291050-03 06

1910