INSTRUCTIONS FOR USE

Anti fog solution alcohol free

CD004-CA

(GB) Anti fog solution alcohol free

(F) Set anti-buée

STERILE R

Irradiation sterilized.

Read IFU
before use

Single use
only

Do not use device
if packaging is
damaged

Rx only

Manufactured for:

CONMED Corporation
525 French Road, Utica, NY 13502
(315) 797-8375 • FAX 1-800-438-3051
Customer Service (USA) 1-800-448-6506
e-mail: info@conmed.com
For International orders or inquiries,
please contact CONMED International Sales
+1 (315) 797-8375 • FAX +1 (315) 735-6235
www.conmed.com

(GB) ENGLISH

◆ Intended Use:

Anti-Fog Solution is intended to be used to prevent "fogging" caused by condensation on the lenses of endoscopic / laparoscopic instruments which are likely to fog during use.

◆ Note:

Sterile unless package is damaged or opened. All contents are Gamma Radiation sterilized.

◆ Instruction for Use:

- ① Remove paper backing from sponge and place sponge onto sterile drape.
- ② Remove cap and apply several drops of Anti-Fog Solution to sponge.
- ③ Moisten the scope with sponge. Then the scope can be used right away.
- ④ Apply another drop of Anti-Fog Solution onto the sponge if the repetition is needed. DO NOT wipe the lens of scope with dry sponge or it may cause damage to the lens.

◆ Cautions/ Warnings:

- This device is Gamma Radiation sterilized and for single use only. Please do not re-use or re-process to avoid irreversible injury such as cross-infection or injury caused by device malfunction.
- Do not use when the package is opened or damaged.
- Store this device in a dry, clean and safe place. Keep the device away from flammable and combustible chemicals or materials. Storage temperature must not exceed 70°C.
- Please follow universal guidelines upon medical waste disposal to avoid any potential cross-infection, environmental contamination or injury caused by sharps waste.
- Eye and skin contact, ingestion or inhalation of the solution may cause mild to slight irritation. Avoid internal application, use around the eyes or other sensitive organs. If contact with eyes, immediately flush eyes with plenty of water for 2-3 minutes. Remove any contact lenses and continue flushing for 15 minutes. Seek medical attention. If contact with skin, remove contaminated clothing including shoes and immediately wash affected area with plenty of soap and water, seek medical attention. Wash contaminated clothing and shoes before reuse. When ingested, wash out mouth with water and keep at rest. Seek immediate medical attention. When inhaled, remove from further exposure. Keep warm and at rest. If cough or other symptoms develop, seek medical attention.

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