

◆ PRODUCT INFORMATION◆

EASI SAFE (DISCARD PAD)

◆ Description:

The EASI SAFE is designed for disposal of surgical blades and needles after procedures. The discard pads are available with either a blister tray or light adhesion blister tray for use in operation rooms.
















◆ Preparation:

- 1) Remove the sleeve containing the EASI SAFE from the peel pouch package.
- 2) Tear the film liner off, put the disposable sharp blade or needle in the glue area, after procedure, then fold it completely.
- 3) To dispose of the EASI SAFE:
The used pad must be discarded in compliance with government regulations for infectious waste disposal.

◆ Cautions/Warnings:

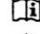- Completely inspect package prior to use. Do not use if the sterile packaging is opened, damaged or wet. Sterility might be compromised.
- Do not use product after expiration date!
- Do not re-use! If re-used, sterility will be compromised.
- Do not re-sterilize!
- "To be disposed according regulations of respective country." or "Dispose used product following local state or federal guidelines."
- "For use by or on the order by a physician, or persons licensed by state law."

Explanation of symbols:

	Caution		Sterilized using Ethylene Oxide
	Do not use if package is damaged		Batch code
	Keep away from sunlight		Date of manufacture YYYY-MM
	Catalogue number		Manufacturer
	Do not re-sterilize		Use by
	Consult instructions for use		Do not reuse
	Keep dry		Authorised representative in the European community.
	Notified body: DNV GL Presafe AS, with CE2460		

Distributed By:



Fannin UK LTD

**Unit 100, Westminster Industrial Estate,
Repton Road, Measham, Swadlincote, DE12 7DT
Tel:01453 874130**


Manufactured By:



Advanced Medical Design Co., Ltd.
4F~5F, No. 29, Wuquan 5th Rd., Wugu Dist.,
24888 New Taipei City, Taiwan



Authorized Address:

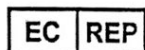

Lotus NL B.V.
Koningin Julianaplein 10, 1e Verd, 2595AA, The Hague,
Netherlands