

# 3M™ Steri-Drape™ Ioban™ 2 Specialty Drapes

## WARNING:

- Do not defibrillate through plastic drapes as arcing or ineffective defibrillation may result.
- Do not resterilize.
- Do not ethylene oxide sterilize.

## Contraindication:


Do not use Ioban 2 Antimicrobial Incise Film on patients with known sensitivity to iodine.

## Indications:

3M Steri-Drape Ioban 2 Specialty Drapes are used to create a sterile field for the surgical procedure. They are intended for external use only.

## Directions for use:

1. Disinfect the surgical or operative site according to hospital policy or physician-prescribed routine.
2. Make sure skin is dry.
3. Apply drape to thoroughly dry intact skin; smooth with a sterile towel.
4. To remove an adhered area, gently peel back at 180 degree angle from the skin. This means folding the adhered area back on itself and maintaining that angle while pulling the film away from the skin (see figure A and B). During the removal process hold onto the adhered area with the thumb and hand as close as possible to the junction of the skin and adhesive. This prevents severe stretching. Skin of elderly patients is typically fragile and thus requires even greater care when applying and removing adhesive products such as drapes.

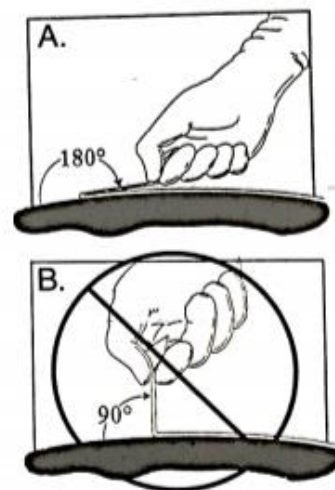

## For all Steri-Drape Ioban products:

- For best results, store at room temperature. Avoid excessive heat and humidity.

## Explanation of symbols



- Sterile unless package is damaged or open.



- This product and package do not contain natural rubber latex.

3M Health Care  
St. Paul, MN 55144-1000  
(U.S.A.) 1-800-228-3957

©3M 2006 All rights reserved.  
3M, Steri-Drape, and Ioban are trademarks of 3M

Issue Date: 2006-09  
34-8707-6248-0

CE 0086

EC REP 3M Health Care  
D-41453 Neuss, Germany