



PRODUCT INFORMATION DATA SHEET

Please read the following information thoroughly before using this device.

Product Ref: LM201, LM202, LM205, LM2025, LM20210, LM280



Description:

This product is available as either a 5mm or a 10mm laparoscopic device and can also be supplied in either Single or Dual Spike versions.

This Suction & Irrigation Set is a packed sterile and is intended for single patient use only.

The functions of both suction and irrigation can be achieved using one single device.

Contraindications:

This device is not intended for use when endoscopic techniques are contraindicated.

Recommended Use:

The Suction & Irrigation Set is intended for use during operative laparoscopy.

Preparation:

1. Whilst adopting an aseptic technique, remove the sleeve containing the Suction & Irrigation set from the sterile packaging.
2. Close the white clamp on the tubing. Remove the safety cap from the spike and insert the spike into the irrigation pump.
3. Please follow instructions provided by suction machine manufacturer for details of connection of the suction adapter to the suction bottle
4. The Suction/Irrigation instrument should be inserted through the trocar. Please note the Suction/Irrigation device is a 5mm instrument and an appropriate trocar diameter must be selected.
5. The Suction option is selected by depressing the White trumpet valve on the device. The irrigation option is selected by depressing the Blue trumpet valve. Please follow instructions provided by suction machine and irrigation pump manufacturer for further information on the safe use of their device.
6. Once the endoscopic procedure is completed, dispose of this device in accordance with local regulations.

Precautions:

1. The Suction & Irrigation Set is designed for single use only, and is not to be re-used or re-sterilized to avoid irreversible injury and device malfunction. This device is EO sterilized.
2. Please ensure compatibility of all laparoscopic instruments used in every procedure to avoid any injury caused as a result of capacitive coupling.
3. Ensure the user has a thorough understanding of the principles of electrosurgery to avoid associated hazards to either patient or user.
4. Do not use if the packaging is opened or damaged.
5. Store this device in a dry, clean and safe place. Keep the device away from flammable and combustible chemicals or materials.
6. Please follow universal guidelines upon medical waste disposal to avoid any potential cross-infection, environmental contamination or injury caused by sharps waste.
7. Federal (U.S.A.) law restricts this device to sale, distribution, and use by, or on the order of a physician.

