

System for collecting pieces of tissue after TURP

DESCRIPTION:

- System for collecting pieces of tissue after TURP
- Fitted with a connector for adapting it to equipment made by:
 - Wolf: REF. E80103
 - Storz: REF. E80203
 - Olympus: REF. E80303
 - ACMI: REF. E80403

INDICATION:

Removal of prostate chips after TURP

CONTRAINICATION

Any known allergies to the medical device materials.

PRECAUTIONS:

- Before use, make sure the cap is screwed down fully on the reservoir.
- Any use other than stated indications is under the responsibility of the physician.
- This type of kits must only be used by experienced and trained professionals
- Reuse of this single use product may create a potential risk to the user. Reprocessing, cleaning, disinfection and sterilization may compromise product characteristics which in turn create an additional risk of physical harm to or infection of the patient.

OPERATING PROCEDURE

See enclosed graphics.

STORE AWAY FROM LIGHT IN A COOL AND DRY PLACE.





DO NOT USE IF PACKAGE IS DAMAGED.

DO NOT RESTERILIZE.

DISCARD PRODUCT AFTER USE.

For Rx Only

Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.

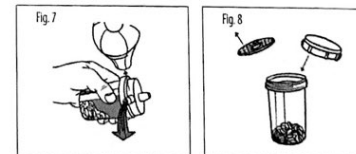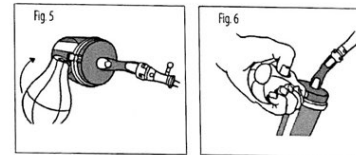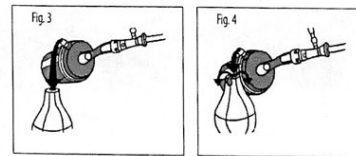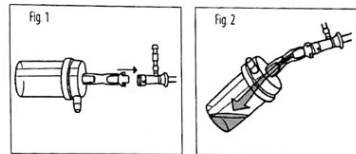


Rx ONLY Caution: Federal (USA) law restricts this device to sale by or on the order of a physician

STERILIZED Sterilized using ethylene oxide


 Do not re-use


 Use By

 Date of manufacture


 Caution: Consult accompanying documents

LOT Batch code

 Keep away from sunlight

 Do not use if package is damaged
Ne pas utiliser si l'emballage est endommagé
Bei beschadigter Verpackung nicht verwenden
Non utilizzare il prodotto se la confezione è danneggiata
No utilizar si el embalaje esta danado
Nao utilize se a embalagem estiver danificada
Niet gebruiken als de verpakking is beschadigd
Miy gypuzumoitte to poivoy ov ij ooukucioia tyti umotrei Oyjao
He mkonuocoiars, ecni yriacocsa nocpekeza
Nie uzywac, jeeli opakowanie jest uszkodzone
V pilpaad polkozeneho baleni nepouzivejte

Ambalay hasarlysa urunu kullatmayin
Ei saa kayttaa, jos pakkauksen vahingottunut
Anvand inte om forpackningen ar skadad
Ma ikke anvendes hvis emballagen er beskadiget
Sikai ikke brukes hvis emballajen er skadet
包装破损不可使用
Ja ne ce isonozasa, ako onakocakata e nocpekeza
Ne hasuzajja, ha a cocmagolias nyitva van vazgy senelt
Negaľma naudoti, jei pakuste pažeista
A nu se utilizeza dacã ambalajul este deteriorat
Ne koristiti ako je paket oštećen
المودة تلفت حال في الاستخدام بضع

 Not manufactured with Natural Rubber Latex
N'est pas fabriqué avec du latex naturel
Nicht mit Naturkautschuklatex hergestellt
Non prodotto con lattice di gomma naturale
Este producto no esta fabricado con goma de latex natural
Nao fabricado com borracha de latex natural
Niet gemaakt uit natuurlijke rubberlatex
Aev eiwei kstokucioajyve oty oovokó latex
Произведен без применения натурального каучукового латекса
Do produkcji nie użyto latektu naturalnego

Vyrobeno bez prirodni gummy
Dođajl kauruk lateks ile ureilmemistir
Valmistettu ilman luonnonkumilateksia
Produkten ar inte tillverkad av aturgummilatex
Indeholder ikke naturgummilatex
Ikke produsert med naturgummilatex
不出天然乳胶制成
He e ppoovzeveno c nateks or ecneteeen kaysjy
Nem tartalmaz természetes gumiálmest
Sudeťyie nera naturalaus lateks
Nu este fabricat cu latex din cauciuc natural
Nije proizvedeno od prirodne lateks gume
بست الطبيعي المطاطي اللاتكس بخر لا

WO Work order
Ordre de fabrication
Arbeitsauftrag
Ordine di lavoro
Orden de trabajo
Ordem de fabrico
Zendingorder
Επιτάγή παραγγελίας
Заказ на изготовление
Numer zamówienia
Pracovní řád
إم امر

Yrmaarays
Arbeitsorder
Produktionsordre
Arbetsordre
工作订单
Работна последователност
Munkaszam
Darbo užsakymas
Comandă de lucru
Radni nalog
التميل امر