

## Instruction for Use

### Yankauer and Suction probes

Remove from packaging

Insert the male connector end of the suction device into female end of connecting tube

If using device with vacuum control, use fingertip to cover vent or slide switch to control suction

### Yankauer and Suction Probes Sets

Remove from packaging

Check all contents are present, and the suction device is securely attached to the tubing.

Connect tubing to suction source

If using device with vacuum control, use fingertip to cover vent or slide switch to control suction



### Warnings:

Do not use if packaging is damaged or open prior to use.

Do not use if you observe any damage to or foreign bodies in the device.

Pennine Yankauer and Suctions Probes devices should not come into direct contact with the central nervous system. (CNS)

TEC-SSP-3.1 v2

CE  
2797